



**PHILIPS**

RESPIRONICS

*DreamWear*

Under the nose  
nasal mask

[milnadumedicalsystem.com](http://milnadumedicalsystem.com)

Rediscover  
**dreams**

# Like wearing nothing at all

Patients can sleep how they want with the freedom and comfort of DreamWear

## Patients can sleep like it's not even there

Do your patients have trouble finding a comfortable sleeping position or feel restricted by CPAP tubing? DreamWear is an innovative mask created to give your patients the choice to sleep how they want.

## The best of pillows and nasal masks


DreamWear's unique design offers many of the benefits of nasal and pillows masks to allow patients to have the best of both mask types. Allowing more freedom of movement and more comfort than their prescribed mask, DreamWear patients feel like they are not wearing a mask at all.\*



### Benefits

- Tubing attachment located at the top of the head to offer freedom of movement during the night
- Prevents red marks, discomfort or irritation in the nostrils or on the nose bridge\*\*
- Users rated DreamWear more stable, more comfortable, and more appealing than their prescribed mask\*
- Soft, flexible silicone frame and fabric wraps provide a soft touch on the patient's cheek

Users told us that DreamWear gave them more freedom to move, had a better fit than their prescribed mask, and comes closest to making them feel like they don't have anything on their face during therapy.\*

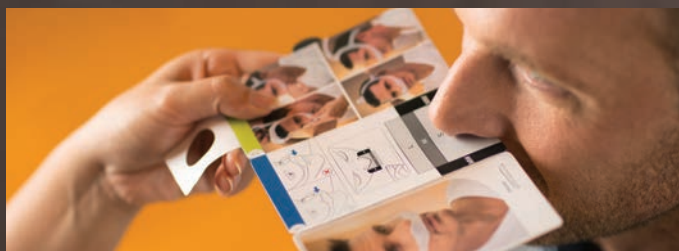

DreamWear cushion size is determined by measuring the length and width of the nose using the simple sizing gauge.\*\*

\*2015 Philips User Preference Questionnaire, data on file.

\*\*Mask does not directly contact the bridge of the nose or nostrils

\*\*\*Medium wide (MW) cushion size uses length and width of the nose





[milnadumedicalsystem.com](http://milnadumedicalsystem.com)

# Get started

To learn more about why DreamWear can be one of the only nasal masks you offer your patients, contact your Philips Respironics representative for more information.

<b>DreamWear fitpak</b>	<b>Part number</b>
DreamWear with headgear, medium frame, all cushion sizes	1116700
DreamWear without headgear, medium frame, all cushion sizes	1116701
<b>DreamWear with headgear, small frame</b>	<b>Part number</b>
Small cushion	1116685
Medium cushion	1116686
Large cushion	1116687
Medium wide cushion	1116688
<b>DreamWear with headgear, medium frame</b>	<b>Part number</b>
Small cushion	1116680
Medium cushion	1116681
Large cushion	1116682
Medium wide cushion	1116683
<b>DreamWear with headgear, large frame</b>	<b>Part number</b>
Small cushion	1116690
Medium cushion	1116691
Large cushion	1116692
Medium wide cushion	1116693
<b>DreamWear without headgear, small frame</b>	<b>Part number</b>
Small cushion	1116710
Medium cushion	1116711
Large cushion	1116712
Medium wide cushion	1116713
<b>DreamWear without headgear, medium frame</b>	<b>Part number</b>
Small cushion	1116705
Medium cushion	1116706
Large cushion	1116707
Medium wide cushion	1116708
<b>DreamWear without headgear, large frame</b>	<b>Part number</b>
Small cushion	1116715
Medium cushion	1116716
Large cushion	1116717
Medium wide cushion	1116718

<b>DreamWear replacement parts</b>	<b>Part number</b>
Small nasal cushion	1116740
Medium nasal cushion	1116741
Large nasal cushion	1116742
Medium wide nasal cushion	1116743
Headgear	1116750
Small frame	1116745
Medium frame	1116746
Large frame	1116747
Fabric wraps	1116754

milnagadumedicalequipment.com



[milnadumedicalsystem.com](http://milnadumedicalsystem.com)



# DreamWear patient fitting guide

Achieving the right fit with DreamWear is easy. Simply follow the instructions below to get started.

**Four cushion sizes (Small, medium, large, medium wide) are available. Please read instructions for use for further fitting information.**

- 

1 Hold the sizing gauge horizontally under the nose and select the cushion size based on where the outer edge of the nostrils and tip of the nose contact the gauge.
- 

2 The cushion opening will be directly under the nostrils. The cushion will hug the nose and leak will be minimal.  
(At no time should the nose be inside the cushion opening. If any part of the cushion overlaps the nose, the cushion is being worn incorrectly.)
- 

3 With the mask assembled, place the cushion under the nose.
- 

4 Position the frame on the top of the head.
- 

5 Pull the headgear over the back of the head.
- 

6 To adjust the mask, peel the headgear tabs away from the fabric. Adjust the straps length. Press the tabs back against the fabric to reattach.  
(Note: Do not overtighten the headgear.)

7



Position the mask until it fits comfortably. When finished, the elbow should rest at the top of the head.

#### Mask frame sizing

The medium (MED) mask frame will comfortably fit most faces. If the MED frame does not fit a patient's face, a small (SM) or large (LG) mask frame might better suit your patient's needs. Call for more information, 1-800-345-6443.

## Part of the Dream Family

DreamWear is part of the Dream Family from Philips Respironics. The Dream Family offers innovative, comprehensive sleep therapy technology like our sleep therapy system, DreamStation, and our patient self-management tool, DreamMapper.



Call for more information, 1-800-345-6443.

milnadumedicalsystem.com



©2015 Koninklijke Philips N.V. All rights reserved.  
Specifications are subject to change without notice.  
Trademarks are the property of Koninklijke Philips N.V.  
(Royal Philips) or their respective owners.

[www.philips.com](http://www.philips.com)

Caution: US federal law restricts these devices to  
sale by or on the order of a physician.

RRDPGH EB 9/10/15 MCI 4106842 PN 1125118  
1010 Murry Ridge Lane, Murrysville, PA 15668  
800-345-6443 • 724-387-4000